

Bowie and Dick Daily Sterilizer Test (B&D Test)

The **Valisafe B&D test pack** fulfils the requirements of both *EN 285* and *EN ISO 17665*

(i) The test is performed at the beginning of each working shift in every vacuum assisted steam sterilizer.

(ii) This non-toxic test is not a test of sterility but a test of the mechanical function of the sterilizer to ensure rapid and even steam penetration into a small porous load.

- Conforms to EN ISO 11140–4
- Independently validated to mirror the performance of the original Towel pack
- Diagnostic capabilities:
 - air removal
 - wet steam
 - super heated steam or presence of non-condensable gases
- Clear and accurate colour change
- Easy to interpret – easy to record
- Non-toxic, single use, disposable
- Lead free

Name:	Daily Bowie & Dick Test Pack, steam penetration
Classification:	EN-ISO 11140-1 (Class 2), EN-ISO 11140-4
Operating temp:	134°C - 137°C
Time:	3.3 min - 3.5 min
Storage conditions:	10°C - 30°C
RH:	30-70%
Correct symbols:	In accordance with EN-ISO 11140-4
Indicator:	Both internal and external, external = process indicator, Internal = diagnostic indicator
Cover	Wrapper
Filtration medium:	Barrier system
Sheet	Silk screen printed
Hospital:	Yes
Department:	Yes
Autoclave no:	Yes
Cycle no:	Yes
Date:	Yes
Operator:	Yes
Supervisor:	Yes
Disp/single use:	Yes
Coated:	Yes
Non-toxic/Lead free:	Yes, according to EC Regulation 1907/2006
Lot no.	Yes
Date:	Yes
Operator:	Yes
Pass/fail result:	Yest
Colour change:	V3501020 pink to brown or darker V3501024 blue to pink

V3501020



Wrapper

Unprocessed Test Pack Indicator Sheet

Reverse of IndicatorSheet

V3501024



Unprocessed Test Pack Indicator Sheet

Wrapper

Order details:

V3501020
V3501024

Daily Bowie & Dick Test Pack Standard
Daily Bowie & Dick Test Pack

Pack of 20
Pack of 20