



Rapid
results!

- Unique & unrivalled weekly residual protein test
- Cost effective
- Accurate results within 10 seconds
- No other test more sensitive without incubation
- HTM 01 05 compliant
- Easy to use
- Clear colour change when protein detected
- Enhanced accuracy
- Shelf life 2 years

Valisafe DentaCheck

The Valisafe Denta-Check protein test is a rapid test that has the ability to detect residual proteins left behind on the surfaces of hard to clean dental and surgical instruments. An easy to read colour change gives you your final result within 10 seconds. Denta-Check is suitable for use with washer disinfectors, ultrasonics and even manual cleaning.

This protein test is based on a dye-binding solution used in clinical chemistry and can detect protein residues within 1ug sensitivity, therefore incubation is not required

Denta-Check meets and exceeds current guidelines for protein testing, as published in HTM 0105 (pg 60 14.4)

Order Details

V3505095

10 vials & short swabs+ 1 positive control test



Pass / Fail

Name:	Valisafe Denta-Check
Conformity:	BS EN ISO 15883, HTM 2030
Time	Result within 10 Seconds
Storage conditions:	Refridgerated (4°C)
Lot no:	yes
Exp date:	yes
Correct symbols:	yes in accordance with the standard
Indicator:	Diagnostic dye-binding vial
Expiry date:	24 months refridgerated 6 months non-refridgerated
Packaging:	10 tests per pack including swabs + 1 protein control test

Quick and easy to use; to determine if protein is present on a surface swab the instrument surface, place the swab in the colour changing solution and swirl for 5 seconds. The swab can then be discarded. If protein is present the reagent will turn blue within 10 seconds. The colour change provides an easy to read result; the higher the level of protein contamination the darker the colour changes to blue.

