



Howarth Trolley Range

Instructions for use

CONTENTS

1.	INTRODUCTION.....	3
2.	SPECIFICATION.....	4
3.	ASSEMBLY & PREPARING FOR USE.....	5
4.	CLEANING.....	5
5.	INSPECTION.....	5
6.	REPLACEMENT/SPARE PARTS.....	6
7.	WARRANTY.....	7

I. INTRODUCTION

Thank you for purchasing a Doherty Howarth trolley. Your Howarth trolley is part of a matching range of furniture, including fixed height couches and privacy screens. Please speak to the Drive Devilbiss Healthcare customer service team for information with regard to the complete range (contact details found on the back of this booklet).





These instructions for use should be read carefully before using your new Howarth Trolley.

In line with our policy of continuous improvement we welcome any suggestions that could lead to product improvement.



Trolley Shown: I525/01/PURPLE

2. SPECIFICATION

	Howarth 01	Howarth 02	Howarth 03	Howarth 04
				
Height	960mm (37.8")	960mm (37.8")	630mm (24.8")	960mm (37.8")
Width	520mm (20.4")	365mm (14.4")	520mm (20.4")	520mm (20.4")
Depth	450mm (17.7")	450mm (17.7")	450mm (17.7")	450mm (17.7")
Trolley Weight	22.2kg (44.5lb)	16.3kg (36lb)	14.6kg (32.2lb)	15.7kg (34.6lb)
Castors	2 Brake + 2 Swivel	2 Brake + 2 Swivel	2 Brake + 2 Swivel	2 Brake + 2 Swivel
Shelves	2	2	2	3
Trolley Frames	1	1	1	1
Plastic drawer sizes and quantity				
Wide, Shallow 440 x 400 x 75mm (17.3 x 15.7 x 3")	4		2	1
Wide, Deep 440 x 400 x 150mm (17.3 x 15.7 x 5.9")	1		1	
Narrow, Shallow 280 x 400 x 75mm (11 x 15.7 x 3")		4		
Narrow, Deep 280 x 400 x 150mm (17.3 x 15.7 x 5.9")		1		
Drawer Ticket Windows	5	5	3	1


All Trolleys come in a range of 5 attractive colours.




White




Purple

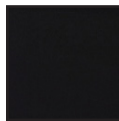

Black

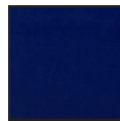

Blue

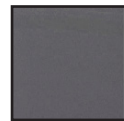

Grey

3. ASSEMBLY & PREPARING FOR USE

Your trolley is delivered fully assembled and ready for use.

1. Remove the trolley carefully from its packaging. (Note: it is advisable for two able bodied people to unpack the trolley due to the weight.)
2. Remove all packing materials that are securing the shelves and drawers in place.
3. Wheel into position and apply the brakes to the front castors.

Please dispose of the packaging in a sensible and environmental fashion.

4. CLEANING

All surfaces should be wiped down with a disposable cleaning cloth, moistened with a mild detergent and diluted in hot water (60°C). Extra care should be taken around areas where excess dirt or dust may gather. Rinse down with clean water and dry off with a paper towel.

Always ensure the trolley is dry before re-using.

Drive Devilbiss Healthcare does not recommend the use of neat bleach as this could mark the painted or plastic surfaces.

5. INSPECTION

The Howarth trolley requires very little maintenance, however if there are any signs of damage or the trolley is not performing as it should, please contact your supplier for assistance.

Inspection points:

- Check there are no loose parts or fixings
- Check no parts appear excessively worn
- Check the castors are clean and free running
- Check the brakes function effectively

6. REPLACEMENT/SPARE PARTS

Spare Part List	Product Code	1525/01 (Standard)	1525/02 (Narrow)	1525/03 (Short)	1525/04 (Basic)
Removable Shelf	1525/013	x		x	x
Removable Narrow Shelf	1525/026		x		
Removable Wide, Shallow Drawer	1525/TRAY/WS	x		x	x
Removable Narrow, Shallow Drawer	1525/TRAY/NS		x		
Removable Wide, Deep Drawer	1525/TRAY/WD	x		x	
Removable Narrow, Deep Drawer	1525/TRAY/ND		x		
Ticket Window	1525/TRAY/TW	x	x	x	x
Brake Castor	CASTOR/010	x	x	x	x
Swivel Castor	CASTOR/011	x	x	x	x

7. WARRANTY

Drive Devilbiss Healthcare Ltd guarantees this product is free from defects in material and workmanship under normal use for 1 year (1 year full parts and labour) from the date of purchase from Drive Devilbiss Healthcare Ltd, its subsidiary companies, authorised dealers and international distributors. Proof of purchase must be presented with any claim. Except as provided herein, Drive Devilbiss Healthcare Ltd, product warranty does not cover damage caused by misuse or abuse, accident, the attachment of any unauthorised accessory, alteration to the product, or any other conditions whatsoever that are beyond the control of Drive Devilbiss Healthcare Ltd. Drive Devilbiss Healthcare Ltd, its subsidiary companies, authorised dealers and international distributors shall have no liability or responsibility to the customers or any other person or entity with respect to any liability, loss or damage caused directly or indirectly by use or performance of the product or arising out of any breach of this warranty, including but not limited to any damages resulting from inconvenience, loss of time, property, revenue, or profit or any indirect, special, incidental or consequential damages, even if Drive Devilbiss Healthcare Ltd, its subsidiary companies, authorised dealers or international distributors has been advised of the possibility of such damages.

In the event of a product defect during the warranty period you should contact your supplier, whether it be Drive Devilbiss Healthcare Ltd, its subsidiary companies, authorised dealers or international distributors who will at their option unless otherwise provided by law; a) correct the defect by product repair without charge for parts and labour b) replace the product with one of the same or similar design or c) refund the purchase price. All replaced parts and products on which a refund is made become the property of Drive Devilbiss Healthcare Ltd. New or reconditioned parts and products may be used in the performance of the warranty service. Repaired or replaced parts and products are warranted for the remainder of the original warranty period. You will be charged for repair or replacement of the product made after the expiration of the warranty period.

This warranty does not cover; a) damage or failure by or attributes to acts of God, abuse, accident, misuse, improper or abnormal usage, failure to follow instructions, improper installation or maintenance, alterations, lightning or other incidence of excess voltage or current, b) any repairs other than those provided by a Drive Devilbiss Healthcare Ltd authorised technician, c) consumables such as fuses, d) cosmetic damage, e) transportation, shipping or insurance costs or f) costs of product removal, installation setup service adjustment or re-installation.

This limited one year warranty gives you specific legal rights and you may also have other rights.

Drive Devilbiss Healthcare Ltd cannot be held responsible for any injury or incident which relates to the use of this Doherty Howarth Trolley in conjunction with accessories manufactured by companies other than Drive Devilbiss Healthcare Ltd.

Drive Devilbiss Healthcare has a policy of continual product improvement and reserves the right to amend specifications covered in this brochure.

No part of this brochure may be reproduced without the written approval of Drive DeVilbis Healthcare Ltd.



CONTACT INFORMATION

Tel: +44 (0) 845 0600 333 Fax: +44 (0) 845 0600 334

Email: info@drivedevilbiss.co.uk

www.drivedevilbiss.co.uk

Drive De Vilbiss Healthcare Ltd., Heathfield Lane, Birkenshaw,
West Yorkshire, BD11 2HW, United Kingdom

A member of the Drive De Vilbiss Group of Companies

