

**Medema Limited.** [3-9 Broomhill Road, London, SW18 4JQ. United Kingdom.]  
Telephone: 0044 (0) 208 8779258, Fax: 0044 (0) 208 8779888  
e-mail: [sales@medema.co.uk](mailto:sales@medema.co.uk), web: [www.medema.co.uk](http://www.medema.co.uk)

## Skin staplers and skin staple removers [3M Precise Vista Skin Staplers]

### 3M™ Precise™ Vista Skin Staplers (Surgical solutions)

- Clear Visibility:** The angled stapler head allows a clear view of the wound area for precise staple insertion.
- Ergonomic Feel:** Fit comfortably in your hand for enhanced control.
- Versatile Staple Release:** Staples can be released from the retractable anvil in any direction.
- Reliable:** Vista's simple design and quality construction assure consistent performance.

The smaller Precise MS skin stapler is also suitable for use on surgical wounds.



## Instructions for use

### Precise Multi Shot skin staplers



1. Approximate and slightly evert skin edges. Hold stapler upright (90°). Place on skin with gentle downward pressure. Increased downward pressure on the skin results in a more deeply embedded staple.



2. To form staple, press trigger firmly and completely.



3. To release device from staple, relax thumb pressure fully.





4. Back the instrument off the staple and lift the stapler away from the skin. Always check staple placement and remove improperly implanted staples.

**Contraindications:**

1. When it is not possible to maintain at least 5mm distance from the stapled skin to underlying vital structures, the use of staplers for skin closure is contraindicated.
2. Not to be used for internal tissue closure.
3. This product is not designed, sold, or intended for use except as indicated.

**Warnings:**

1. Single patient use only.
2. Do not reprocess or resterilize, this may compromise the functional integrity of this product.

[Product Code] Product Name	Image	Unit Price
<p>[ <b>ST3996</b> ] <span style="float: right;"><b>NEW</b></span>                      3M™ Precise™ Vista Disposable Skin Stapler – A unit/box of 6 staplers. Each stapler contains 15 staples.</p>		<p>£64.47 (Ex. VAT)</p>
<p>[ <b>ST3995</b> ] <span style="float: right;"><b>NEW</b></span>                      3M™ Precise™ Vista Disposable Skin Stapler – A unit/box of 6 staplers. Each stapler contains 35 staples.</p>		<p>£64.47 (Ex. VAT)</p>