

Medema Limited. [3-9 Broomhill Road, London, SW18 4JQ. United Kingdom.]
Telephone: 0044 (0) 208 8779258, Fax: 0044 (0) 208 8779888
e-mail: sales@medema.co.uk, web: www.medema.co.uk

Surgical Clippers [3M Surgical Clipper 9661 with Pivoting Head]

3M™ Surgical Clipper 9661


The Surgical Clipper 9661 takes the clipping technology to “state of the art”. The ergonomic design and unique pivoting head makes it light and easy to manoeuvre on even the most awkward body contours.

- Universal for body and head removal
- Cuts hair in both raking and pushing motions
- Removes all types of hair, fine or coarse
- Cutting blade on top of the stationary blade, away from the skin, so there is minimal risk of nicks and cuts
- Minimum 40-minute run time after fully charging the battery (8 hours)
- Snap on/snap off blade, reducing the time between clippings
- Single-use blades, reducing the risk of cross contamination between patients
- Easy-to-clean handle, just wipe with a disinfectant or rinse under warm running water (never immerse the clipper)





How to

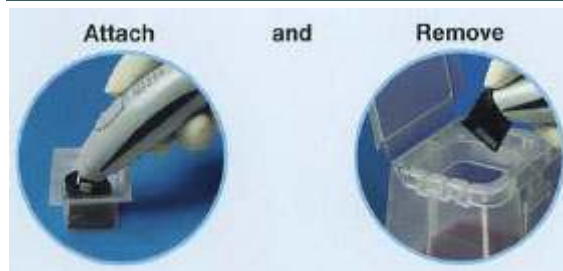

Quick

Single use blade assemblies are designed to snap on and pop off, saving time and helping prevent cross contamination.



Easily cleaned



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
Simply wipe the clipper handle with an alcohol wipe, or disinfectant, or rinse the head of the clipper handle under running water.



Clipping

You can push or pull, whichever you find best. Position the clipper blade between 15° and 30° to the skin surface and gently push or pull across.

1 Alexander, Wesley et al, "The Influence of Hair Removal Methods on Wound Infections", Archives of Surgery, 1983; 118:347-352
 2 Shannon, John Jr. et al, "Preoperative Skin Preparation and Wound Infection", Infections in Surgery, June 1985
 3 Sellick, JA Jr. et al, "Surveillance of Surgical Wound Infections Following Open Heart Surgery", Infection Control and Hospital Epidemiology, 1991; 12:591-596
 4 Guideline for Prevention of Surgical Site Infection, Centers for Disease Control, U.S. Public Health Service, Revision 1999 Vol.20 N.4
 5 Saving Lives, a delivery programme to reduce Healthcare Associated Infections including MRSA, Department of Health, 2006.

[Product Code] Product Name	Image	Unit Price
[SC9661] 3M™ Surgical Clipper 9661 with Pivoting Head		£92.49 (Ex. VAT)
[SC9660] 3M™ Single-use Blades for Surgical Clipper with Pivoting Head - A unit/box of 50 Blades.		£149.24 (Ex. VAT)
[SC9665] 3M™ Charger for Surgical Clipper with Pivoting Head		£72.00 (Ex. VAT)

