

## Surgical Clippers [3M Surgical Clipper 9671 with Fixed Head]

### 3M™ Surgical Clipper 9671

#### New, unique cordless/corded operation

- Universal for body and head removal
- Removes all types of hair, fine or coarse, wet or dry
- Operates with or without a cord
- Clipper blade assembly is fixed (non rotating) – easy attachment, easy removal
- Ergonomic design fits comfortably in hand, features non-slip grip
- Improved, state-of-the-art performance: easier to use for dependability
- NiMH Battery charges continuously during corded operation; LED light indicates charging mode
- Cutting blade on top of the stationary blade, away from the skin, so there is minimal risk of nicks and cuts
- Cuts hair in both raking and pushing motions
- Single-use blades reduces cross-contamination and are easy to snap on and pop off
- Provides approximately 45 minutes of cordless run time from a single 12-hour recharge
- Easy-to-clean handle, just wipe with a disinfectant or rinse under warm running water (never immerse the clipper)




CORDLESS



CORDED




**How to**

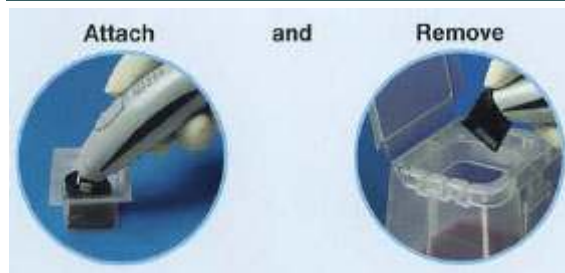

**Quick**

Single use blade assemblies are designed to snap on and pop off, saving time and helping prevent cross contamination.

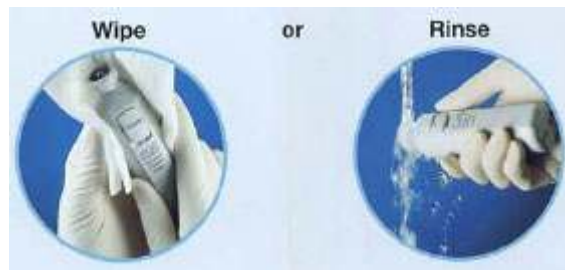

**Easily cleaned**

Simply wipe the clipper handle with an alcohol wipe, or disinfectant, or rinse the head of the clipper handle under running water.



**Clipping**

You can push or pull, whichever you find best. Position the clipper blade between 15° and 30° to the skin surface and gently push or pull across.

- 1 Alexander, Wesley et al, "The Influence of Hair Removal Methods on Wound Infections", Archives of Surgery, 1983; 118:347-352
- 2 Shannon, John Jr. et al, "Preoperative Skin Preparation and Wound Infection", Infections in Surgery, June 1985
- 3 Sellick, JA Jr. et al, "Surveillance of Surgical Wound Infections Following Open Heart Surgery", Infection Control and Hospital Epidemiology, 1991; 12:591-596
- 4 Guideline for Prevention of Surgical Site Infection, Centers for Disease Control, U.S. Public Health Service, Revision 1999 Vol.20 N.4
- 5 Saving Lives, a delivery programme to reduce Healthcare Associated Infections including MRSA, Department of Health, 2006.

[Product Code] Product Name	Image	Unit Price
[ SC9671 ] 3M™ Surgical Clipper with fixed head		£89.51 (Ex. VAT)
[ SC9670 ] 3M™ Surgical Clipper single use blades - fixed head – A unit/box of 50 blades		£148.50 (Ex. VAT)
[ SC9675 ] 3M™ Surgical Clipper charging unit for fixed head surgical clipper		£36.72 (Ex. VAT)
[ SC9669 ] 3M™ Surgical Clipper wall mount for fixed head surgical clipper		£5.36 (Ex. VAT)
[ SC9676 ]		£14.99

3M™ Surgical Clipper charge stand for fixed head surgical clipper



NO IMAGE  
AVAILABLE

(Ex. VAT)